





# Reproduzierbarkeit von V-3D mit V-Classic-Farben im Kiss-Konzept:







V-3D-Farben	Opaker	Dentin-Mischung				Schneide
		Dentin-Grundmassen		Power Chroma-Zusatzmassen		
		 Portionierer groß	 Portionierer groß	 Portionierer klein	 Portionierer klein	
1 M 1	B1	B1	-	-	-	S1
1 M 2	A1	A1	A2 (kein)	-	-	S1
2 L 1,5	B2	B2	-	-	-	S1
2 L 2,5	B2	B2	B3	-	-	S1
2 M 1	D2	D2	-	-	-	S2
2 M 2	A2	A2	A3	-	-	S2
2 M 3	B3	B3	-	PC 1	PC 3	S4
2 R 1,5	A2	A2 (2x)	-	PC 1	PC 4	S2
2 R 2,5	A2	A3	A2	PC 1	PC 4	S2
3 L 1,5	C2	C2	-	-	-	S5
3 L 2,5	B3	B3	-	-	-	S4
3 M 1	C1	C1	C2	-	-	S1
3 M 2	A3	A3	-	-	-	S3
3 M 3	B4	B4	-	PC 3	PC 5	S6
3 R 1,5	D3	D3 (2x)	-	PC 5	-	S4
3 R 2,5	A3,5	A3,5 (2x)	-	PC 3	PC 5	S3
4 L 1,5	C3	C3	C4 (klein)	-	-	S5
4 L 2,5	A4	A4 (2x)	-	PC 4	PC 6	S5
4 M 1	C3	C3	C4	-	-	S5
4 M 2	A3,5	A3,5 (2x)	-	PC 4	PC 6	S3
4 M 3	A4	A4	-	PC 4	PC 6	S5
4 R 1,5	A3,5	A3,5	-	PC 5 (2x)	-	S3
4 R 2,5	A4	A4	-	PC 4	PC 5	S5
5 M 1	C4	C4	-	PC 4	PC 6	S6
5 M 2	A4	A4	-	PC 4	PC 6	S5
5 M 3	A4	A4	-	PC 4 (groß)	PC 6 (groß)	S5

Die hier angegebenen Zuordnungen sind Richtwerte und dienen ausschließlich als Anhaltspunkt. Abweichungen der Farben sind möglich. Alle Angaben sind von uns sorgfältig erstellt und geprüft worden, werden jedoch ohne Gewähr weitergegeben.

[www.kiss-keramik.de](http://www.kiss-keramik.de)

# Reproducibility of V-3D-Shades within the Kiss Concept.



V-3D-SHADES	Opaque	Dentine mixture				Incisal
		Dentine basic powders		Power Chroma additional powders		
		 Portioner large	 Portioner large	 Portioner small	 Portioner small	
1 M 1	B1	B1	-	-	-	S1
1 M 2	A1	A1	A2 (kein)	-	-	S1
2 L 1,5	B2	B2	-	-	-	S1
2 L 2,5	B2	B2	B3	-	-	S1
2 M 1	D2	D2	-	-	-	S2
2 M 2	A2	A2	A3	-	-	S2
2 M 3	B3	B3	-	PC 1	PC 3	S4
2 R 1,5	A2	A2 (2x)	-	PC 1	PC 4	S2
2 R 2,5	A2	A3	A2	PC 1	PC 4	S2
3 L 1,5	C2	C2	-	-	-	S5
3 L 2,5	B3	B3	-	-	-	S4
3 M 1	C1	C1	C2	-	-	S1
3 M 2	A3	A3	-	-	-	S3
3 M 3	B4	B4	-	PC 3	PC 5	S6
3 R 1,5	D3	D3 (2x)	-	PC 5	-	S4
3 R 2,5	A3,5	A3,5 (2x)	-	PC 3	PC 5	S3
4 L 1,5	C3	C3	C4 (small)	-	-	S5
4 L 2,5	A4	A4 (2x)	-	PC 4	PC 6	S5
4 M 1	C3	C3	C4	-	-	S5
4 M 2	A3,5	A3,5 (2x)	-	PC 4	PC 6	S3
4 M 3	A4	A4	-	PC 4	PC 6	S5
4 R 1,5	A3,5	A3,5	-	PC 5 (2x)	-	S3
4 R 2,5	A4	A4	-	PC 4	PC 5	S5
5 M 1	C4	C4	-	PC 4	PC 6	S6
5 M 2	A4	A4	-	PC 4	PC 6	S5
5 M 3	A4	A4	-	PC 4 (large)	PC 6 (large)	S5

The values listed here are intended for orientation only and should be regarded only as guidelines. The shades could differ. We have compiled and checked all values and other data with care. However, we cannot under any circumstances be liable for your results.

[www.kiss-ceramics.com](http://www.kiss-ceramics.com)



