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Extreme™

Verblendkeramikmassen

Instructions for use

Mode d'emploi

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DeguDent

A Dentsply International Company

Extreme™

D	Gebrauchsanweisung	4
GB	Instructions for use	5
F	Mode d'emploi	6
I	Istruzioni per l'uso	7
E	Instrucciones para el uso	8
P	Instrução de uso	9
PL	Informacje dotyczące obróbki	10
CZ	Pokyny pro zpracování	11
DK	Bearbejdelsesindikationer	12
GR	Υποδείξεις επεξεργασίας	13
H	Feldolgozási útmutató	14
HR	Napomene za obradu	15
J	加工について	16
LT	Apdorojimo nuorodos	17
LV	Norādes sakarā ar pārstrādi	18
N	Bruksanvisningen	19
NL	Instructies voor de bewerking	20
RC	加工说明	21
ROK	가공처리 주지사항	22
RUS	Рекомендации по работе с материалом	23
S	Beredningsanvisningar	24
TR	İşlem talimatları	25
UAE	تعليمات بخصوص التحضير	26

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Supplement to the instructions for use for Cercon® ceram kiss, Duceram® Kiss, Duceragold® Kiss and Duceratin® Kiss

Extreme™ veneering ceramics are supplementary dentines and opaques for the Cercon® ceram kiss, Duceram® Kiss, Duceragold® Kiss and Duceratin® Kiss veneering ceramics.

1. General instructions on processing Extreme veneering ceramics:

- For information on indications for use, contraindications, warnings, precautions, adverse reactions, technical data, transport and storage conditions, explanations of symbols on the product labels and combinable liquids, please consult the enclosed instructions for use for the respective Kiss line of veneering ceramics (Cercon® ceram kiss, Duceram® Kiss, Duceragold® Kiss or Duceratin® Kiss).
- Extreme veneering ceramics can be used as a replacement for or mixed with the opaques or dentines of the respective Kiss line of veneering ceramics. To complete the buildup, the Kiss line of veneering ceramics (Cercon ceram kiss, Duceragold Kiss, Duceram Kiss or Duceratin Kiss) must also be used.
- Extreme veneering ceramics are fired at the same temperatures as the classic opaques or dentines.
- For firing recommendations, please refer to the instructions for use of the respective Kiss line of veneering ceramics. Follow the recommendations for the corresponding framework material.

2. Shade concept:

The A3 shade is the center of the entire shade concept. Based on this, two shades each have been added at each end of the line of A shades (A1– A4): on one hand E0 and E0.5 in the region of the very bright shades (bleaching shades) and on the other hand E5 and E6 on the high-chroma side. Also based on practical experience, all shades (A and E) can be shifted in four directions – toward the more highly chromatic (intensive), yellow, red and gray. In this way, the shade concept follows the intuitive sequence of visual shade matching.

3. Four-step shade matching:

- Start out with the A3 shade
- Best match between E0 and E6, central line
- Choose the spot shade (red, yellow, intensive, gray)
- Best match within the spot line

4. Set composition:

The small supplementary set includes two additional basic dentine shades (E0 and E6 plus their appropriate opaques) and four mixed dentine shades (spots) for the individual control of the basic dentine shade (intensive, yellow, red, gray).

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* All dentines are based on the standard Kiss 1 : 1 mixing ratio.

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DeguDent GmbH